POLSKI

ENGLISH

36-060 Głogów/Miłostowski, ul. Piaski 47
ul. Truskawkowa 17, Widna Góra, 37-500 Jarosław, POLAND
tel.: +48 (016) 621 42 20
tel.: +48 (016) 621 41 35, fax.: +48 (016) 621 42 13
e-mail: biuro@reh4mat.com, Mailing address
www.reh4mat.com; www.reh4mat.pl
The company holds the ISO 13485
quality management system certificate
DATA AKTUALIZACJI / THE UPDATE DATE OF THE INSTRUCTION: 09.10.2015

REH4MAT

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KNEE BRACE

ATTENTION: BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTION COMPLETELY AND CAREFULLY AND CHECK IF BRACE'S ELEMENTS AND SEAMS ARE SEEDY. ONLY UNDAMAGED DEVICE AND ITS CORRECT APPLICATION ARE VITAL TO THE PROPER FUNCTIONING OF IT. SAVE THE PRODUCT INFORMATION, TO BE ABLE TO REREAD THE INSTRUCTION LATER.

PRODUCT DESCRIPTION:

Knee brace is the excellent solution for effective knee joint therapy in case of injury or other conditions. Knee braces are appreciated by specialists (physicians, physical therapist) and patients. Knee brace improves motor function of knee. By maintaining of uniform temperature and elastic pressure it has analgesic properties. Reh4Mat products are innovative and are carefully made of the high quality materials for you. Our key objective is customer's satisfaction. In case of any questions feel free to contact with us by e-mail: biuro@reh4mat.com.

PURPOSE OF USE:

- degenerative conditions,
- arthritis,
- pain syndrome,
- ligaments rupture (ACL),
- ligaments reconstruction (ACL).

IMPORTANT:

- Proper use of the knee brace requires diagnosis by physician or physical therapist.
- Optimal brace functioning can only be achieved by selecting the right size. The Reh4Mat knee brace must be fitted by an authorized vendor, physician or educated physiotherapist.
- It is necessary to control using the brace, especially in children, people with memory dysfunction, psychiatric and people with sensory disturbances.
- The skin in place, where the brace touches should be clean (the creams or ointments should not be used).
- Is it necessary to control the skin's condition and in case of skin lesions occurring is it necessary to contact with physician.
- In case of skin lesions (epidermis scrapes, scatches or eczema) using of the braces is allowing after dressing applying.
- Chronic using the brace may cause limitation of the active and passive movement in joints. If there are no contraindications, it is recommended to exercise several times

during the day. Exercises are selected individually for each patient by a master's degree or technician rehabilitation as physician has recommended.



- In case of varicose veins, Deep Vein Thrombosis (DVT), lower limb oedema the using of brace is allowing only after contact with physician.
- Use of the brace during the sleeping should be done only if this was prescribed by your doctor.

CARE AND MAINTENANCE:

- Wash by hands in warm water with soap in temperature of 30 C,
- Squeeze carefully,
- Do not iron,
- Do not chlorinate,
- Do not dry clean,
- Do not tumble dry,
- Protect from moisture,
- Protect from high temperatures,
- Dry away from sources of warmth.

CAUTION: IF THE DEVICE HOLDS LATERAL SPLINTS, SHELLS AND IT IS POSSIBLE TO DISMANTLE THEM WITHOUT ANY TOOLS, BEFORE WASHING REMOVE THE SPLINTS.

ATTENTION: Pain is always the sign of warning. If the pain maintains or increases it is advisable to consult your doctor or physical therapist.
ATTENTION: BRACE SHOULD BE USED ONLY BY ONE PATIENT. USING OF THE DEVICE NOT ACCORDING TO THE MANUAL MAY CAUSE SERIOUS COMPLICATIONS.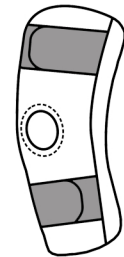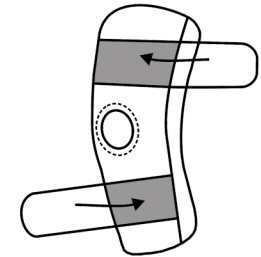

4 army
MILITARY BRACES