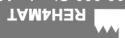
POLSKI

ENGLISH

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WRIST BRACE

ATTENTION: BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTION COMPLETELY AND CAREFULLY AND CHECK IF BRACE'S ELEMENTS AND SEAMS ARE SEEDY. ONLY UNDAMAGED DEVICE AND ITS CORRECT APPLICATION ARE VITAL TO THE PROPER FUNCTIONING OF IT. SAVE THE PRODUCT INFORMATION, TO BE ABLE TO REREAD THE INSTRUCTION LATER.

PRODUCT DESCRIPTION:

Wrist brace is an excellent solution for effective wrist therapy in case of injury or other conditions. Braces are appreciated by specialists (physicians, physical therapist) and patients.

Wrist brace provides wrist support with stabilization and compression.

Reh4Mat products are innovative and are carefully made of the high quality materials. Our key objective is customer's satisfaction. In case of any questions feel free to contact with us by e-mail: biuro@reh4mat.com

PURPOSE OF USE:

- pain syndrome,
- inflammations,
- injuries.

IMPORTANT:

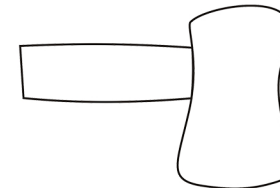
- Proper use of the brace requires diagnosis by physician or physical therapist.
- Optimal brace functioning can only be achieved by selecting the right size. The Reh4Mat brace must be fitted by an authorized vender, physician or educated physio-therapist.
- It is necessary to control using the brace, especially in children, people with memory dysfunction, psychiatric and people with sensory disturbances.
- The skin in place, where the brace touches should be clean (the creams or ointments should not be used).
- Is it necessary to control the skin's condition and in case of skin lesions occurring it is necessary to contact with physician.
- In case of skin lesions (epidermis scrapes, scathes or eczema) using of the braces is allowing after dressing applying.
- Chronic using the brace may cause limitation of the active and passive movement in wrist. If there are no contraindications, it is recommended to exercise several times during the day. Exercises are selected individually for each patient by a master's degree or technician rehabilitation as physician has recommended.

- In the case of poor blood circulation in the limbs, a brace may cause hyperaemia. Avoid applying the splint too tightly in the beginning.
- Use of the brace during the night should be done only if this was prescribed by your doctor.


CARE AND MAINTENANCE:

- Wash by hands in warm water with soap in temperature of 30 C,
- Slice carefully,
- Do not iron,
- Do not chlorinate,
- Do not dry clean,
- Do not tumble dry,
- Protect from moisture,
- Protect from high temperatures,
- Dry away from sources of warmth.

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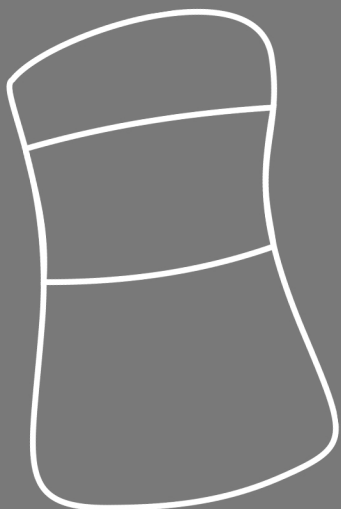ATTENTION: Pain is always the sign of warning. If the pain maintains or increases it is advisable to consult your doctor or physical therapist.

ATTENTION: BRACE SHOULD BE USED ONLY BY ONE PATIENT. USING OF THE DEVICE NOT ACCORDING TO THE MANUAL MAY CAUSE SERIOUS COMPLICATIONS.